



PHYSIOTHERAPY TABLE CLAVE. REF. 2252B



High performance bed with breathing hole suitable for physiotherapy and osteopathy treatments. It is divided into 6 sections that allow the easy regulation of the table in several positions. The leg supports are independent. Equipped with 3 motors that control height, backrest and central module movements. Sturdy metal frame with retractable wheels to facilitate its movement. White PU upholstery of high quality and easy cleaning on high density padding.



TECHNICAL DATA

DIMENSIONS: 207*68*61-88 cm

PACKING SIZE: 186 x 72 x 65 cm

GROSS WEIGHT / NET WEIGHT: 86 kg / 67 kg

VOLTAGE: 100V-120V/ 220V—240V

FREQUENCY: 50/60 HZ

UPHOLSTERY: High quality PU

NUMBER OF MOTORS: 3

HANDCONTROLLER: YES

BREATHER HOLE: YES

ROLL HOLDER: NO

ADJUSTABLE ARMREST: YES (see technical drawing)

WHEELS: YES, RETRACTABLE

HEATING: NOT AVAILABLE



COLORS AVAILABLE: WHITE, GREEN, BLACK

MAX. HEIGHT LEGREST EXT.: 143 cm



MAXIMUM LOAD: 175 KG



BACKREST ADJUSTMENT



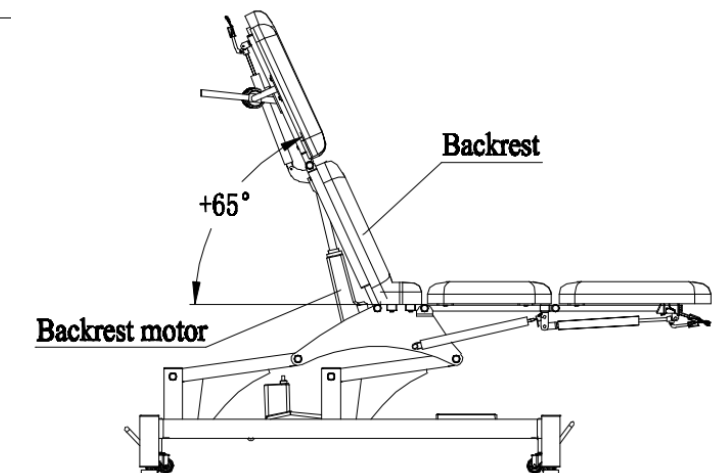
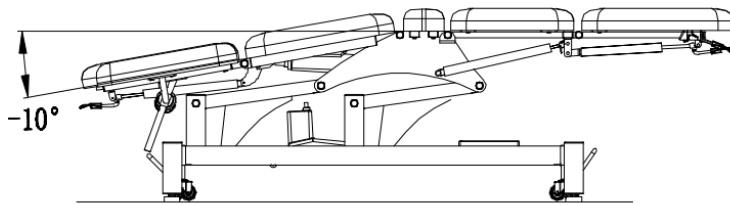


HEADREST ADJUSTMENT

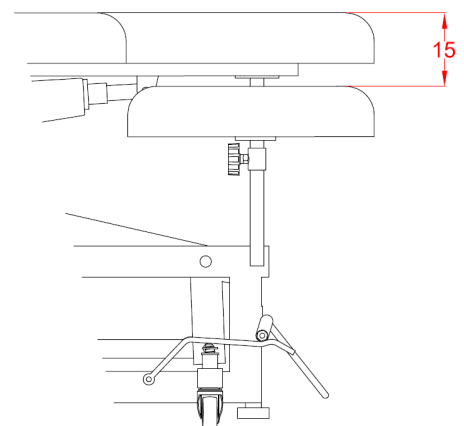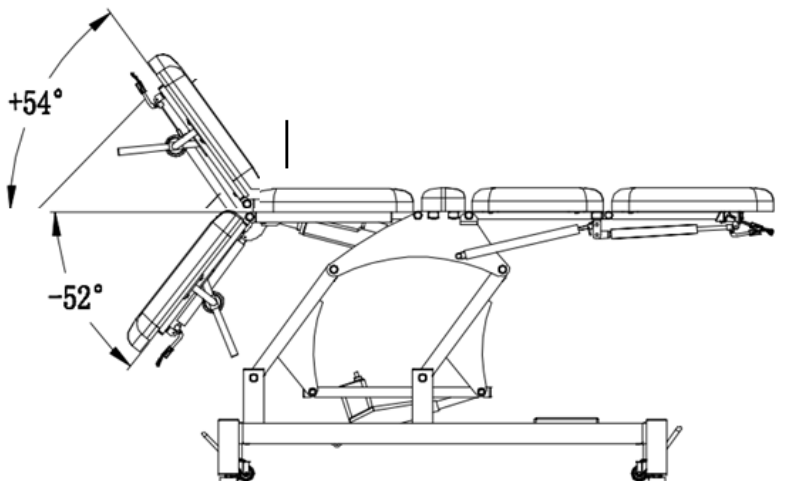



FOOTREST ADJUSTMENT

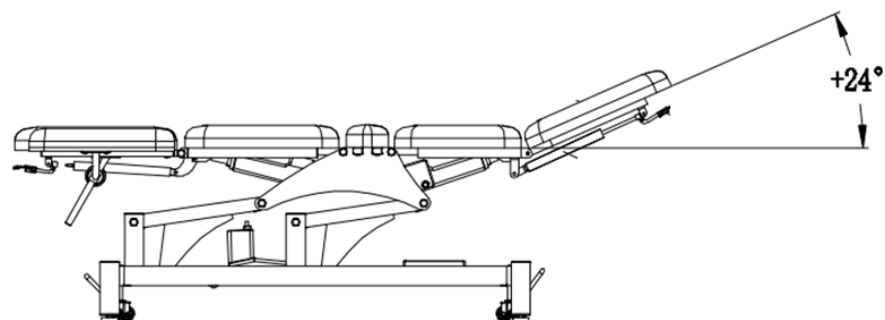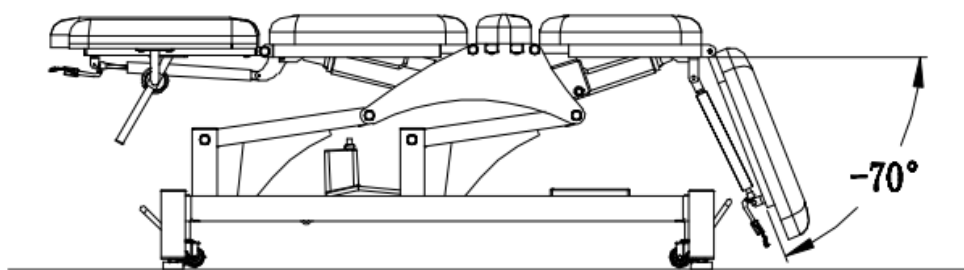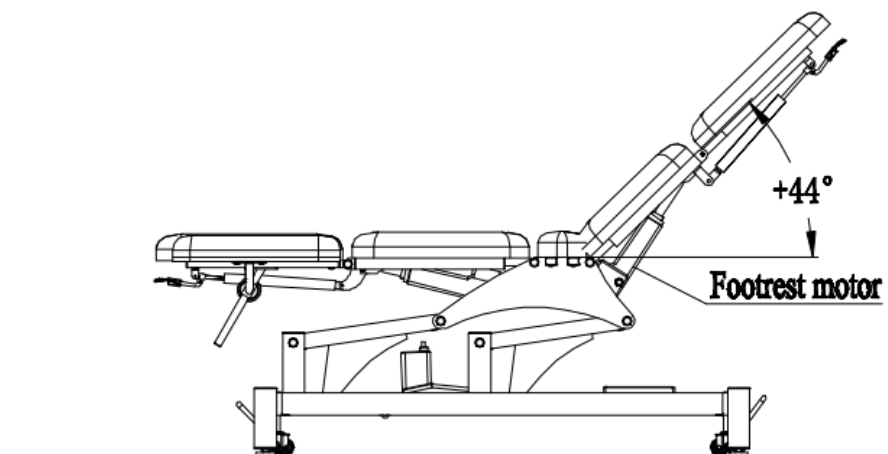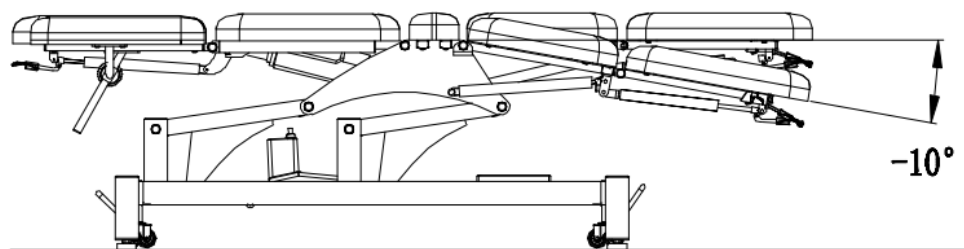



CAUTIONS

- 1) To avoid coming into collision between the different parts, the backrest shouldn't go down at the same time that the headrest when the bed is in the lowest position.
- 2) If the bed is in the lowest position and the backrest is placed horizontally (as in the picture below), the headrest should only go down until -22° .
- 3) If you are moving the bed into the lowest position, please be careful if the footrest is by -10° to avoid coming into collision with the floor.

